

# TR 9650

## Mobile Bath Lift System

Hygiene Equipment  
Designed for People

### Quick guide for the Stretcher Lift - Combi Lift



Open lid:



**Working Load Limit: 150 kg (330 lbs)**

Position	Description		
1	Support column	12	Hygiene slot
2	Chassis Base	13	"TR "Protection cover
3	Transfer handle	14	Stretcher attachment and position setting bars
4	Chassis leg	15	Safety belts
5	Positive brake castors, 100mm Twin	16	Transport grip
6	Straight steering castors, 100mm Twin	17	Stabilizing bar
7	Foot control for raising	18	Arm/Back support
8	Foot control for lowering	19	Drive belt
9	Stainless Steel column	20	Low battery audible tone
10	Hygiene Seat	21	Battery charger input
11	Stretcher side parts	22	Emergency Stop

#### Head office


TR Equipment AB, Sweden  
Phone: +46 140 38 50 60  
info@treqipment.se  
www.treqipment.com

#### Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900  
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386  
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



**TR EQUIPMENT**

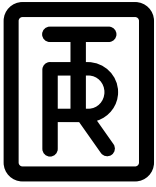

# TR 9650

## Mobile Bath Lift System

Hygiene Equipment  
Designed for People

### Quick guide for the Stretcher - Combilift



- Only qualified staff, who understand all operating and safety procedures may operate the equipment.
- Safety belts must be securely fastened around patient at all times and patients feet should rest firmly on the stretcher pad.
- Never leave the patient unattended.
- Castor brakes must be securely locked on the Lifter before transferring a patient on to the chair/stretcher.
- The patient must be positioned comfortably on the chair/stretcher, with folded rotating arm/stabilizing bar.
- Make sure that all parts that come in contact with the patient are intact and without sharp edges.
- Ensure that the Lift can be moved, raised or lowered safely to avoid any pinch injuries to patient.
- Operate the Lift carefully when transporting patients through narrow corridors or over irregular uneven surfaces.
- The TR battery charger is the only one approved for charging.
- Do not use battery charger in wet or damp areas - potential shock hazard.
- It is recommended that the batteries are charged daily or whenever Lifter is not in use. Batteries can not be overcharged.
- Check that castor brakes and straight steering are free from damage.
- Be sure that all details are in place and free from damage and all screws are in place.
- Pads must be kept well maintained in place.
- Lift and chair/stretcher must be inspected visually for sign of wear. Lift should not be used if defects are noted. Prompt service is required.

**Warning:** Never use the Lifter in any way that varies from the safety precautions and operating procedures detailed in the operating manual. The equipment must not be operated by anyone until the user has fully read and understands the instructions in the operating manual.

#### **Safety autostop mechanism:**

If the chair/stretcher hits an obstacle when lowering, the Lift will safely stop automatically. To release it, just press the "up" right arrow foot control. Same procedure if the lifting belt is slacking. If lifting belt fails, the chair/stretcher will stop/lock automatically on the column. Call for technical assistance.

**For further information see Operating manual**

#### **Head office**

TR Equipment AB, Sweden  
Phone: +46 140 38 50 60  
info@treqipment.se  
www.treqipment.com

#### **Group Subsidiaries**

TR Group Inc, US, Phone: +1 (800) 752 6900  
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386  
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



**TR EQUIPMENT**